Myriad MatrixTM

1

Soft Tissue Bioscaffold

 **ARO BIOSURGERY LTD.**
2 Kingsford Smith Place
Airport Oaks, Auckland 2022
New Zealand

www.aroabio.com

AROTM, Myriad MatrixTM and
ARO A E C MTM are trademarks of
Aroa Biosurgery Ltd.

PI.9326.03 | ©July 2022

Instructions For Use

EN Product description

Myriad Matrix™ is an intact extracellular matrix (ECM) derived from ovine (sheep) forestomach tissue. This advanced ECM scaffold is non-reconstituted collagen, thus it retains the innate biological structure and function of the native ECM associated macromolecules including elastin, fibronectin, glycosaminoglycans and laminin.

Myriad Matrix transforms into a soft conforming sheet, which is incorporated into the surrounding tissue over time. The device is supplied sterile and dry in a variety of sizes and thicknesses, which can be trimmed to size or applied whole to meet the individual patient's needs.

Myriad Matrix is available by prescription only.

Not made with natural rubber latex.

Intended use

Myriad Matrix is intended for applications in plastic and reconstructive surgery or to cover, protect, and provide a moist wound environment. The device may be fixed, via sutures, staples, or tacks to the surrounding tissue, if desired.

Indications for use

Myriad Matrix is indicated for implantation to reinforce soft tissue where weakness exists in patients requiring soft tissue repair or reinforcement in plastic and reconstructive surgery, or for management of the following wounds:

- Partial and full thickness wounds
- Pressure ulcers
- Venous ulcers
- Diabetic ulcers
- Chronic vascular ulcers
- Tunneled/undermined wounds
- Surgical wounds (donor sites, grafts, post Moh's surgery, post laser surgery, podiatric, wound dehiscence)
- Trauma wounds (abrasions, lacerations, second-degree burns, and skin tears)
- Draining wounds

Contraindications

- **Myriad Matrix** is derived from an ovine (sheep) source and should not be used on patients with known sensitivity to ovine (sheep) derived material
- **Myriad Matrix** is not indicated for use on third degree burns

Risk statements

PRECAUTIONS

- **Myriad Matrix** may not have sufficient strength to support stresses encountered in some ventral hernias or large-area, body-wall repairs. **Myriad Matrix** is not intended to augment breast reconstruction.
- Do not apply **Myriad Matrix** in the presence of uncontrolled clinical infection, acute inflammation, excessive exudate, or bleeding
- **Myriad Matrix** is supplied sterile. Do not use if the pouch is damaged
- Discard the device if mishandling has caused possible damage or contamination.
- Single use product. Do not re-sterilize. Discard all unused portions. Do not reuse. Device reuse may contribute to wound infection.
- Minimize manipulation of **Myriad Matrix** during rehydration and placement.
- Always handle **Myriad Matrix** using aseptic technique.

POTENTIAL COMPLICATIONS

The following complications are possible. If any of the complications occur and cannot be resolved, consider removal of the device:

- Infection
- Acute or chronic inflammation
- Allergic reaction
- Seroma formation

Storage

Myriad Matrix should be stored at room temperature in a clean and dry area.

Directions for use

These recommendations are designed to serve only as a general guideline. They are not intended to supersede institutional protocols or professional clinical judgment concerning patient care.

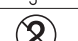
Myriad Matrix can be used in conjunction with compression therapy and negative pressure wound therapy under the supervision of a health care provider.

1. Rehydrate the device in sterile saline or sterile Lactated Ringer's solution for a minimum of 5 minutes.
2. Trim the device to fit, if necessary, providing an allowance for overlap. Multiple sheets can be used. Position the device to achieve maximum contact between the device and surrounding tissue. To facilitate cell migration and tissue ingrowth, an overlap of 3-5 cm with healthy well-vascularized tissue is suggested.
3. If desired, fix in place using standard techniques, avoiding excess tension. Alternately, the device can be secured using an appropriate secondary dressing.
4. For best results when used for wound management, ensure Myriad Matrix remains hydrated.
5. Discard any unused portions according to institutional guidelines for medical waste.

Symbols glossary

Symbols contained in **ISO 15223-1 Medical devices** – Symbols to be used with medical device labels, labeling and information to be supplied – Part 1: General requirements are indicated below.

	5.1.1	Manufacturer
	5.1.3	Date of manufacture
	5.1.4	Use by/Expiration date
	5.1.5	Lot number
	5.1.6	Catalog number

	5.2.3	Sterilized using ethylene oxide
	5.2.6	Do not re-sterilize
	5.2.8	Do not use if packaging is damaged
	5.2.12	Double sterile barrier system
	5.3.4	Keep dry
	5.4.2	Do not reuse
	5.4.3	Consult instructions for use
	5.4.8	Contains biological material of animal origin
	5.7.7	Medical Device
	-	Number of units
	-	Number of layers
Rx Only	-	Caution: Federal (USA) law restricts this device to sale by or on the order of a physician.

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